

Product Name: Venturi Mask Kit

Function :

A mask is an aerosol placed on the patient's nose, mouth or tracheal incision to supply oxygen or aerosol. This product is supplied with a connector that provides different oxygen concentrations for those who require different oxygen concentrations.

Instruction for Use :


1. Place the oxygen mask over the patient's face and place the elastic band over the back of the head to adjust the proper tightness.
2. Connect the oxygen short tube to the mask joint, and connect the short tube to the other end according to the joint of the required concentration.
3. Connect the adapter to the concentration connector and the oxygen conduit to the concentration connector.
4. Connect the oxygen tube to the flow meter and adjust the oxygen to the appropriate flow rate for patient use.

Warning :

1. Please confirm the integrity of the packaging and product before use. Do not use if it is damaged or damaged.
2. The product is used for a single patient.
2. Please make sure all parts are connected before use.
3. This instruction is only a general indication, and it still needs to be used by professional medical personnel.
4. The maximum recommended life of this device is 7 days.
5. Dispose of in accordance with the local medical waste disposal regulations.

Specification : Adult / Child

Valid period: 3 years

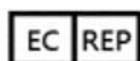

Obelis s.a
Bd. Général Wahis 53 1030 Brussels, BELGIUM



Manufacturer : DA CHUNG MEDICAL CO., LTD.
Address : No. 44, Te-Xing 6th Rd, Dong-Shan
Township, Yi-Lan 26950 Taiwan.
Website : <http://www.dc-med.com.tw>

REV01