

Product Name: Nebulizer Kit

Function :

For the upper respiratory tract drug spray treatment. Its spray particles are 5~10um.

Instruction for Use :


1. Place the spray mask over the patient's face and place the elastic band behind the head to adjust the proper tightness.
2. Open the top of the spray can, inject the medicine according to the doctor's instructions, and then install the upper cover. When the operation, the spray can needs to be used vertically, open the gas supply source switch, set the gas flow rate 6~8 (LPM), that is, spray; , flow setting and use time, please follow the doctor's instructions and connect with the spray mask to achieve therapeutic effect.

Warning :

1. The product is used by a single patient.
2. Please make sure all parts are ready to use before use.
3. This instruction is only a general indication, and it still needs to be used by professional medical personnel.
4. Please confirm the appearance of the package and the product before use. If it is damaged or damaged, please do not use it.
5. The maximum recommended life of this device is 7 days.
6. Dispose of in accordance with the local medical waste disposal regulations.

Specification : Child / Adult, Oxygen Tube Length: 2M

Valid period: 3 years

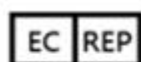

Obelis s.a
Bd. Général Wahis 53 1030 Brussels, BELGIUM



Manufacturer : DA CHUNG MEDICAL CO., LTD.
Address : No. 44, Te-Xing 6th Rd, Dong-Shan
Township, Yi-Lan 26950 Taiwan.
Website : <http://www.dc-med.com.tw>

REV 01