

Product Name: Hi-Oxygen Mask Kit

Function :

An oxygen mask is a device placed on a patient's nose, mouth or tracheal incision to supply oxygen or aerosol.

Instruction for Use :


1. Connect the oxygen mask to the oxygen mask air bag connector.
2. Place the oxygen mask over the patient's face and place the elastic band over the back of the head to adjust the proper tightness.
3. Connect the oxygen tube to the flow meter and adjust the oxygen to the appropriate flow for patient use.

Warning :

1. This product is limited to single use, please do not reuse.
2. The use of the product must be used by a medical professional or by a doctor.
3. Dispose of the used waste according to local medical waste disposal regulations.
4. Do not store the product in the environment of sun, rain or radiation, and avoid damage to the extrusion.
5. Please confirm the integrity of the packaging and product before use. Do not use if it is damaged or damaged.
6. The maximum recommended life of this device is 7 days.
7. Dispose of in accordance with the local medical waste disposal regulations.

Specification : Adult/child

Valid period: 3 years

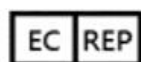

Obelis s.a
Bd. Général Wahis 53 1030 Brussels, BELGIUM



Manufacturer : DA CHUNG MEDICAL CO., LTD.
Address : No. 44, Te-Xing 6th Rd, Dong-Shan
Township, Yi-Lan 26950 Taiwan.
Website : <http://www.dc-med.com.tw>

REV 01