

Product Name: Bubble Humidifier

Function :

A humidifier is a device that adds water vapor to a breathing gas for respiratory therapy or other medical treatment. The vapor generated by this device can spread around the patient, allowing it to breathe vapor during normal breathing, and a whistle warning sound when the pressure is released.

Instruction for Use :

1. Add water to the appropriate level of the humidifier. The water volume should not exceed the upper limit of the water level and the top cover of the humidifier should be locked.
2. Connect the humidifier to the flow meter (or the appropriate device) and the other port of the humidifier can be connected to a suitable oxygen nasal cannula or mask for patient use.
3. Adjust the appropriate oxygen flow for use by the patient.


Warning :

1. The product is used by a single patient.
2. Please confirm the integrity of all parts and packaging before use. If there is any doubt or damage, please do not use this product.
3. This instruction is only a general indication, and it still needs to be used by professional medical personnel.
4. Please confirm the appearance of the package and the product before use. If it is damaged or damaged, please do not use it.
5. The maximum recommended life of this device is 7 days.
6. Dispose of in accordance with the local medical waste disposal regulations.

Specification :

1. Pressure relief: 2PSI/4PSI/6PSI
2. Interface: DISS threaded interface
3. Capacity: 300/300/400 ML
4. Oxygen tube interface: 6mmOD

Valid period: 3 years

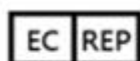

Obelis s.a
Bd. Général Wahis 53 1030 Brussels, BELGIUM



Manufacturer : DA CHUNG MEDICAL CO., LTD.
Address : No. 44, Te-Xing 6th Rd, Dong-Shan
Township, Yi-Lan 26950 Taiwan.
Website : <http://www.dc-med.com.tw>

REV 01