

Product Name: Nebulizer Handheld Kit

Function :

For the upper respiratory tract drug spray treatment. Its spray particles are 5~10um.

Instruction for Use :

1. Open the top of the spray can, inject the medicine according to the instructions of the doctor, and then install the cover.
2. Connect the left and right connectors of the T connector to the mouthpiece and the short tube, and connect the bottom to the spray can.
3. The oxygen conduit is connected to the spray can and the flow meter respectively.
4. When operating, spray cans should be used vertically. Turn on the gas supply switch and set the gas flow rate of 6~8 (LPM), which means spray, dosage, flow setting and use time. Please follow the doctor's instructions.

Warning :

1. The product is used by a single patient.
2. Please make sure all parts are ready to use before use.
3. This instruction is only a general indication, and it still needs to be used by professional medical personnel.
4. Please confirm the appearance of the package and the product before use. If it is damaged or damaged, please do not use it.
5. The maximum recommended life of this device is 7 days.
6. Dispose of in accordance with the local medical waste disposal regulations.

Specification :

18mmOD small nebulizer set

Valid period: 3 years



REV 01